




POSSIBLE FIXATIONS

of the eSense Temperature sensor



You can simply hold the sensor, fixate it with the velcro tape (included), use an adhesive tape or you can use our Finger Clip. The adhesive tape is available in any pharmacy and the finger clip is available in our online shop at : www.mindfield-shop.com

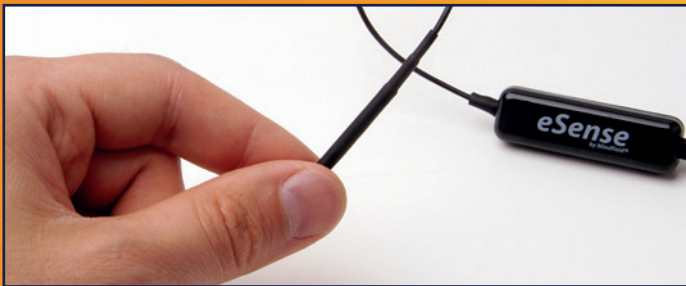

Holding

No additional equipment necessary, simply hold the sensor between your index finger and the thumb.



Velcro Tape

(included)

Included with the eSense Temperature comes a velcro tape. You can use the Velcro tape to fixate the sensor to your finger.




Adhesive Tape

(medical tape, additional)

Simply fixate the sensor to a finger or the palm of your hand with an adhesive tape. You can purchase an adhesive tape at any pharmacy.

OUR RECOMMENDED FIXATION

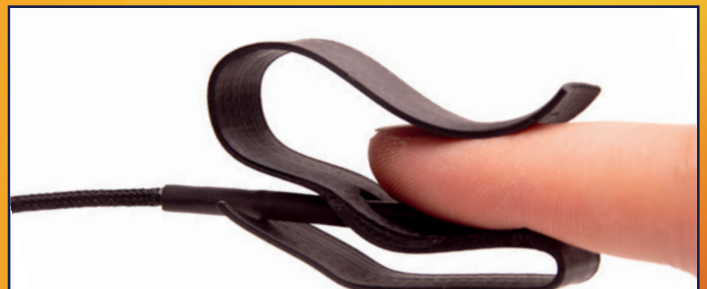

Finger Clip

(additional)

This is our recommended fixation. The Finger Clip never sticks since no glue is used. The open design makes also sure that there won't be any heat accumulated. You can order the Finger Clip in our online store.

[ORDER NOW >>](#)



A Better State of Mind